


Pelvic-Tilt™

Adjustable Displacement Device INSTRUCTIONS FOR USE

1. Place the Pelvic-Tilt™ Device on the table.
2. While adjusting the patient into position, center the device underneath the patient's hips so the pillow will inflate under the hip to be elevated and the flap will remain under the other hip.
3. Be sure the tubing faces outwards, away from the patient's hip.
4. Turn the stopcock to the "INFLATE" position and inflate the device by squeezing the bulb. (An optional adapter is available for tourniquet box inflation.)
5. Inflate until the patient's hip is tilted at the proper angle, then turn the stopcock to the "HOLD AIR" position.
6. Recheck the angle as often as necessary during the procedure. If patient is experiencing discomfort, readjust or reinflate the device. If there is excessive tilting, use the "DEFLATE" position to release some air slowly.
7. When the procedure is completed and the patient is ready to be transferred, deflate the device until the patient rests in the supine position.

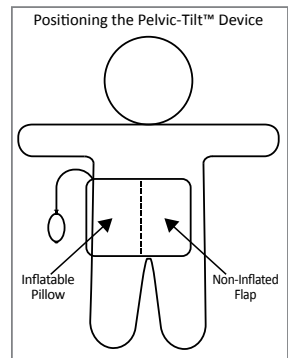

Device is limited use. Discard according to facility's protocol when soiled or stained, or if there is evidence of damage.

To clean, wipe device with one of the following:

- Mild detergent or 70% isopropyl alcohol on a slightly damp cloth/towel
- Quaternary disinfectant wipe
- 0.5% bleach solution on a slightly damp cloth/towel

Allow device to dry thoroughly before reuse.



NEXT Medical Products Company
45 Columbia Road, Branchburg, NJ 08876 USA
800-458-4254 | Fax: 908-218-3715
customerservice@nextmedicalproducts.com
www.nextmedicalproducts.com, www.orcomfort.com

ISO 13485



LATEX-FREE



Annie Bauer Confort
4 av. de la Petite Bermone 06270
Villeneuve Loubet, France

Made in USA

U.S. Patent #6,327,724