

Delgado™

Adjustable Inflatable Post Support

INSTRUCTIONS FOR USE

1. After adjusting the patient into the desired position on the operating table, place the Delgado™ Inflatable Post on an OR table post using the attached sleeve.
2. Center the device on the post as desired to stabilize the patient and to distribute the pressure on the patient's body so as to minimize the likelihood of tissue damage.
3. Be sure the tubing faces backwards, away from the OR table post.
4. Turn the stopcock to the "INFLATE" position and inflate the device by squeezing the bulb. (An optional adapter is available for tourniquet box inflation.)
5. Turn the stopcock to the "HOLD AIR" position when pressure has reached the desired level.
6. When the procedure is completed and the patient is ready to be transferred, turn the stopcock to the "DEFLATE" position.

Device is limited use. Discard according to facility's protocol when soiled or stained, or if there is evidence of damage.

To clean, wipe device with one of the following:

- Mild detergent or 70% isopropyl alcohol on a slightly damp cloth/towel
- Quaternary disinfectant wipe
- 0.5% bleach solution on a slightly damp cloth/towel

Allow device to dry thoroughly before reuse.



NEXT Medical Products Company
45 Columbia Road, Branchburg, NJ 08876 USA
800-458-4254 | Fax: 908-218-3715
customerservice@nextmedicalproducts.com
www.nextmedicalproducts.com
www.orcomfort.com

ISO 13485



LATEX-FREE



Annie Bauer Confort
4 av. de la Petite Bermone 06270
Villeneuve Loubet, France

Made in USA

U.S. Patent #7,802,333

70126B