

INTENDED USE

- The Metal Max+ Combi Laryngoscope is used to illuminate the oropharynx and trachea to assist with endotracheal intubation or allow examination of the oral cavity.
- This laryngoscope handle is supplied with 6V button cell batteries pre-installed and ready to use.

WARNINGS

- Tracheal intubation requires specialist training. Only competent trained personnel should use this device.
- Fixed hinge combination laryngoscope set. Blade and handle are not interchangeable.
- PROACT Medical advocate that intubation is only attempted with a full alternate plan in place for the event of difficult intubation due to equipment failure or physiological complications. This should include the availability of back-up devices and alternative means of ensuring ventilation.
- Reuse of medical devices intended for single use has associated dangers of cross-patient infection and may compromise the integrity, functionality or clinical efficacy of the device.
- In the event that one of these devices is involved in a serious clinical incident, please report it to your local competent authority.

PRE-USE CHECKS

Do not use this device unless the following checks have been performed prior to each use

- Check device and batteries for completeness, sharp edges, damage, flaws or loose parts.
- Engage and disengage the laryngoscope twice to confirm operation. Ensure consistent light when force is exerted on the blade tip in all directions.

INSTRUCTIONS FOR USE

- Select appropriately sized device for patient.
- To switch on, pull blade up and lock into position.
- Light will activate automatically.
- Examine / intubate trachea following current accepted medical guidelines.
- Remove batteries after use by unscrewing bottom cap. Batteries should be recycled after use in accordance with local regulations.

METAL MAX+ COMBI LARYNGOSCOPE BLADE & HANDLE

| Part Number | Style | Size | Batteries Supplied | Blade Length (mm) | Blade Width (mm) |
|-------------|-------------|------|----------------------|-------------------|------------------|
| MMCB90MAC0 | Macintosh | 0 | 4 x 1.5V Button Cell | 61 | 8.5 |
| MMCB90MAC1 | Macintosh | 1 | 4 x 1.5V Button Cell | 75 | 8.5 |
| MMCB90MAC2 | Macintosh | 2 | 4 x 1.5V Button Cell | 93 | 11.5 |
| MMCB90MAC2A | Macintosh | 2 | 2 x 3V Button Cell | 93 | 11.5 |
| MMCB90MAC3 | Macintosh | 3 | 2 x 3V Button Cell | 110 | 13.5 |
| MMCB90MAC35 | Macintosh | 3.5 | 2 x 3V Button Cell | 122 | 13.5 |
| MMCB90MAC3S | Macintosh | 3 | 2 x 3V Button Cell | 110 | 13.5 |
| MMCB90MAC4 | Macintosh | 4 | 2 x 3V Button Cell | 135 | 14 |
| MMCB90MAC4S | Macintosh | 4 | 2 x 3V Button Cell | 135 | 14 |
| MMCB90MAC5 | Macintosh | 5 | 2 x 3V Button Cell | 156 | 14 |
| MMCB90MIL00 | Miller | 00 | 4 x 1.5V Button Cell | 44 | 10.5 |
| MMCB90MIL0 | Miller | 0 | 4 x 1.5V Button Cell | 55 | 10.5 |
| MMCB90MIL1 | Miller | 1 | 4 x 1.5V Button Cell | 81 | 10.5 |
| MMCB90MIL15 | Miller | 1.5 | 4 x 1.5V Button Cell | 100 | 10.5 |
| MMCB90MIL2 | Miller | 2 | 4 x 1.5V Button Cell | 131 | 13 |
| MMCB90MIL2A | Miller | 2 | 2 x 3V Button Cell | 131 | 13 |
| MMCB90MIL3 | Miller | 3 | 2 x 3V Button Cell | 170 | 13 |
| MMCB90MIL4 | Miller | 4 | 2 x 3V Button Cell | 182 | 17 |
| MMCB90LT3 | Lever Tip | LT 3 | 2 x 3V Button Cell | 110 | 13.5 |
| MMCB90LT4 | Lever Tip | LT 4 | 2 x 3V Button Cell | 135 | 14 |
| MMCB90CP00 | Cardiff PRO | 00 | 4 x 1.5V Button Cell | 55 | 8 |
| MMCB90CP0 | Cardiff PRO | 0 | 4 x 1.5V Button Cell | 71 | 9 |
| MMCB90CP1 | Cardiff PRO | 1 | 4 x 1.5V Button Cell | 77 | 10 |

METAL MAX+ LARYNGOSCOPE COMBI BLADE AND HANDLE



SINGLE USE



DO NOT USE IN AN MRI ENVIRONMENT



LATEX FREE



PROACT Medical Ltd

9-13 Oakley Hay Lodge, Great Folds Road, Oakley Hay Business Park
Corby, Northants, NN18 9AS, UK
Tel: +44 (0)1536 461981 E-mail: sales@proactmedical.co.uk

Distributed in North America by:
NEXT Medical Products Company
45 Columbia Road, Branchburg, NJ 08876

IFU 301-V2-2019-04

customerservice@nextmedicalproducts.com

70237A