

# INSTRUCTIONS FOR USE



**STERILE | R**



## Clear Image Singles® STERILE Ultrasound Scanning Gel

Remove the outer peel pouch (if applicable) in accordance with your facility's protocol.

1. To open the single patient packet, grasp the end and tear off along the perforation to expose the built-in dispensing spout.
  2. Apply gel prior to scanning by gently squeezing out the desired amount. Adjust the pressure of your fingers on the packet to regulate the flow of gel coming through the spout. If desired, use your thumb and index finger to apply pressure at the packet's bottom, then slide fingers toward the open end to expel the gel.
  3. After use, dispose of opened packets according to your facility's protocol.
- Clear Image Singles® STERILE Ultrasound Scanning Gel can be used for single-patient use on or near broken skin, where sterility is indicated.
  - Sterile unless package is damaged or opened.

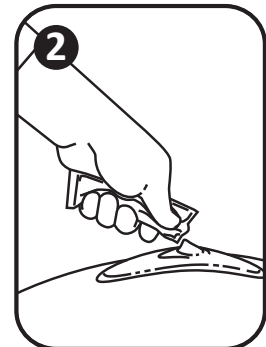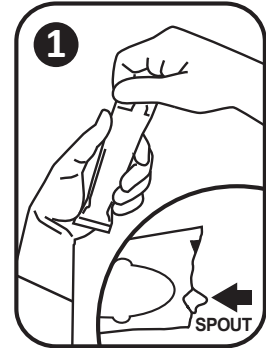

### Storage:

- Store away from direct sunlight to preserve viscosity.
- Recommended storage temperature: 32°F to 105°F (0°C to 41°C).

Any serious incident that has occurred in relation to this device should be reported to NEXT Medical Products.

### Symbols:

<b>REF</b>	Part Number		Use by Date		
<b>LOT</b>	Lot Number		Date of manufacture (production date)	<b>CE</b>	Complies with European Union regulations
<b>STERILE   R</b>	Sterilized using irradiation		Manufacturer		
	Do not reuse		Temperature limits	<b>EC   REP</b>	Authorized representative in the European Union



NEXT Medical Products Company  
45 Columbia Road, Branchburg, NJ 08876 USA

Web: [nextmedicalproducts.com](http://nextmedicalproducts.com)  
Order Desk: 800-458-4254, Order Fax: 908-218-3715  
[customerservice@nextmedicalproducts.com](mailto:customerservice@nextmedicalproducts.com)

**CE** 2460  
ISO 13485:2016

**EC | REP**

Contek, S.S. Appia km 196k500  
81041 Vitulazio, Italy

70158D (DEC 12-2022)