



DATA SHEET - INSTRUCTIONS FOR USE

Clear Image® Ultrasound Scanning Gel

DESCRIPTION

- Ultrasound coupling media for diagnostic and therapeutic imaging.
- Non-sterile; for external use only.
- Provides optimal sound velocity and acoustic impedance.
- Manufactured utilizing a proprietary method to minimize the presence of air bubbles in the gel for improved image clarity.
- Water-based, water-soluble formula; clear, non-staining.
- Contains no irritating dyes, perfumes, parabens or surfactants.
- Bacteriostatic.
- Available in two viscosities: high and medium
- Made in USA.

INSTRUCTIONS FOR USE

- Open bottle. Squeeze out desired amount of gel.
- Apply to the skin and/or to the transducer probe prior to scanning.
- Close cap after use.
- It is recommended to write the date opened on the bottle label. Discard opened bottles in accordance with facility protocol (typically after 28-30 days).
- Gel can be removed using a dry and/or wet paper towel or cloth.
- Gel may be warmed before use. Please check temperature of gel before applying to patient's skin.

TRANSDUCER COMPATIBILITY

- Contains no oils, alcohols, silicones, fragrances or other ingredients that may cause harm to the transducer.
- Approved by major diagnostic ultrasound equipment manufacturers.

ORDERING INFORMATION

Description	Part No.	Viscosity	Contents
8 oz. squeeze bottle with twist steeple cap	02-912	High	12/Case
	05-912	Medium	12/Case
8 oz. squeeze bottle with flip-top cap	02-812	High	12/Case
	05-812	Medium	12/Case
5-Liter container	02-405	High	4/Case (includes two empty 8 oz. refill btls/cs)
	02-105	High	1/Case (includes one empty 8 oz. refill btl/cs)
	05-405	Medium	4/Case (includes two empty 8 oz. refill btls/cs)
	05-105	Medium	1/Case (includes one empty 8 oz. refill btl/cs)
60-Gram Tube	02-6012	High	12/Case
	05-6012	Medium	12/Case

FUNCTION

- Couples diagnostic ultrasound between the transducer and the patient.
- Provides increased acoustic transmission and reduced scattering from air bubbles.

CONTRAINDICATIONS

- Not intended for use with defibrillators.
- Not intended for use in invasive ultrasound procedures or on non-intact skin.



STORAGE

- Store away from direct sunlight to preserve viscosity.
- Recommended storage temperature: 32°F to 105°F (0°C to 41°C).
- Shelf life is five years.

SYMBOLS

REF Part number **LOT** Lot number

UDI Unique device identifier **MD** Medical device

 No natural rubber latex  Use by date

 Manufacturer  Temperature limits

CE Complies with European Union regulations

EC REP Authorized representative in the European Union



NEXT Medical Products Company, 45 Columbia Road, Branchburg, NJ 08876 USA
800-458-4254, 908-722-4549, Fax: 908-218-3715, Web: nextmedicalproducts.com
Email: customerservice@nextmedicalproducts.com

EC REP Contek, S.S. Appia km 196k500, 81041 Vitulazio, Italy

CE 2460

ISO 13485:2016

70001I (OCT 27-2023)