






INSTRUCTIONS FOR USE

Clear Image Singles® Ultrasound Scanning Gel

1. To open the single patient packet, grasp the end and tear off along the perforation to expose the built-in dispensing spout.
 2. Apply gel prior to scanning by gently squeezing out the desired amount. Adjust the pressure of your fingers on the packet to regulate the flow of gel coming through the spout. If desired, use your thumb and index finger to apply pressure at the packet's bottom, then slide fingers toward the open end to expel the gel.
 3. After use, dispose of opened packets according to your facility's protocol.
- Gel can be removed from patient's skin using a dry and/or wet paper towel or cloth.
 - To warm gel before using, place packets in warmer; check temperature of gel before applying to patient's skin.
 - Clear Image Singles Gel is non-sterile and for external use only. It is not for use as an electromedical conductor.
 - Recommended for single patient use.

Storage:

- Store away from direct sunlight to preserve viscosity.
- Recommended storage temperature: 32°F to 105°F (0°C to 41°C).

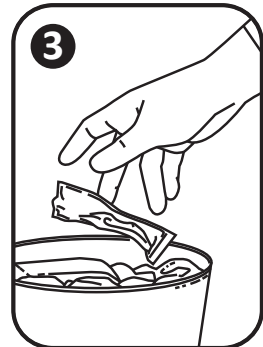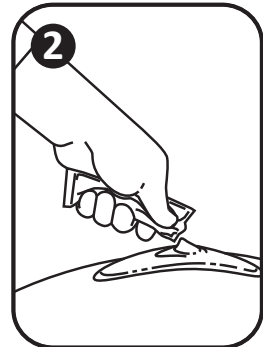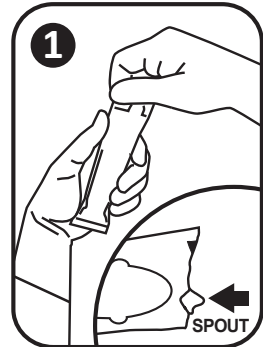

NEXT Medical Products Company
45 Columbia Road, Branchburg, NJ 08876 USA
www.clearimagesingles.com
Order Desk: 800-458-4254, Order Fax: 908-218-3715
Email: customerservice@nextmedicalproducts.com

ISO 13485

EC REP

Contek, S.S. Appia km196k500
81041 Vitulazio, Italy

Made in USA

70064E